Gebruiksaanweisung

Instructions for use

Mode d'emploi

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medimex 

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English

mediPlugs⁺ are available in 12 different sizes:

36, 37, 38, 39, 40, 41,
42, 43, 44, 45, 46, 47

in accordance with
EN ISO 20347:2012

Footwear manufacturer

medimex GmbH
In den Fritzenstücker 9-11
65549 Limburg
fon +49.6431.7302 300
fax +49.6431.7302 235
www.medimex.de

mediPlugs⁺ – Important notes!

antistatic OR shoe with orthopaedic insole

Please read the following information and notes carefully.

Dear Customer,

Congratulations on acquiring your mediPlugs⁺ – the perfect footwear for your day in the OR! You have chosen the optimum hygienic solution with utmost wearing comfort. To ensure that you enjoy your mediPlugs⁺ for a long time, please heed the following instructions:

Use and usage instructions

1. Please make sure you have chosen the right size by putting on the shoes.
2. Please examine the mediPlugs⁺ for any visible defects (intactness of the material, the sole, the strap if applicable) prior to wearing them.
3. mediPlugs⁺ should be washed prior to their first use.
4. mediPlugs⁺ can be washed and disinfected along with the insole in suitable cleaning machines up to 60° and using any of the detergents recommended for your appliance. medimex recommends the Miele models G 7882, G 7882 CD, G 7892, G 7835, G 7836, PG 8535, PG 8536, G 7824, G 7826, PG 8528, PG 8821, PG 8822. Prior to cleaning please attach the insole in the hole on the inside of the shoe intended for this purpose.
5. Always allow the mediPlugs⁺ to dry in such a way that the water can drain. You should preferably position them upright so that water cannot collect inside the shoe. Please do not dry the footwear by placing it on the radiator or similar device.
6. It is necessary to let cool down completely the mediPlugs⁺.
7. Please do not use knives or other sharp objects to treat mediPlugs⁺! The footwear does not require any additional care. Please do not use any shoe or plastic care products!

It is mandatory to wear shoes with safety requirements if it is to be expected that foot injuries may occur. These may result from pushing and jamming, objects tipping over, falling or rolling down, stepping into pointy or sharp objects, hot and caustic liquids, among other causes. Please also heed the regulations of your professional organization. If the shoe appears to be damaged, you must discontinue using it. Using or attaching additional elements that were not part of the product from the start may impair the protective function and thus your safety. The following warranties apply to shoes that are in good condition. medimex does not assume any responsibility for improper uses or uses that are not listed in these usage instructions.

Standards

The CE marking on the shoe means that the shoe meets the basic requirements of the European regulation 2016/425 on personal protective equipment.

The mediPlugs⁺ footwear satisfies the categories and requirements of the following standard:
EN ISO 20347:2012

Personal protective equipment – Professional footwear

Moreover, the following standards apply:

EN ISO 20344:2011

Personal protective equipment – Test methods for footwear

EN ISO 13287:2007

Personal protective equipment – Footwear – Test method for slip resistance

The product was tested by TÜV Rheinland LGA Products GmbH, Tillystrasse 2, D-90431 Nürnberg, ph.: +49 (0)911 655 5225, fax: +49 (0)911 655 5226, email: service@de.tuv.com, URL: www.tuv.com, ID of the notified body: 0197.

Marking

Every professional shoe must be clearly and permanently marked with the following information: a) size, b) manufacturer's symbol, c) manufacturer's type designation, d) year and at least the quarter of manufacture, e) reference to the international standard, f) the symbols from the table below indicating the intended protection and/or appropriate categories, if applicable:

Requirements concerning professional footwear

(Excerpt from the EN ISO 20347:2012 standard)

You are equipped with a pair of professional shoes. The EN ISO 20347:2012 label which you find on this product guarantees:

The requirements concerning professional foot-

wear are met and a good balance between comfort and sturdiness is achieved.

Additional information about antistatic footwear

Antistatic footwear should be used when it is necessary to reduce electrostatic charge build-up by discharging the static electricity in order to exclude the danger of igniting, for instance, flammable substances or vapours through sparks, and if the danger of electric shock due to an electrical device or voltage conducting parts cannot be entirely excluded. **However, it should be pointed out that antistatic footwear cannot provide sufficient protection against electric shock, as it only builds up resistance between the ground and the foot.** If the danger of electric shock cannot be categorically excluded, additional measures must be taken to avoid this danger. Such measures and the following tests should be part of the routine accident prevention programme at the workplace.

Experience has shown that for antistatic purposes, the discharge path through a product should have an electrical resistance of less than 1,000 MΩ throughout the product's life. A value of 100 kΩ is specified as the lowest limit for the resistance of a new product to provide limited protection against dangerous electric shock or ignition due to a defect in an electric appliance when dealing with up to 250 V. However, it should be observed that under certain conditions, the shoe does not offer sufficient protection; for this reason the user of the footwear should always take additional protective measures.

The electrical resistance of this type of footwear may change considerably when the shoe is bent, soiled or when moisture has built up. This shoe may not fulfil its intended function when worn under wet conditions. For this reason it is necessary to ensure that the product is able to carry out its intended function of discharging static electricity and offering a certain degree of protection during its working life. Therefore it is recommended that the user perform an in situ electrical resistance test if necessary and to repeat this test on a regular basis at short intervals.

Footwear of classification I may absorb moisture when worn for an extended period and may become conductive in moist and wet conditions.

If the shoe is worn under conditions where the sole material is contaminated, the user should check the electrical properties of the footwear every time before entering a dangerous area.

In areas where antistatic footwear is worn, the resistance of the ground should be such that the protective function provided by the shoe is not eliminated.

When using the footwear, no insulating components should be inserted between the inner sole of the shoe and the user's foot. If an insole is put between the inner sole of the shoe and the user's foot, the connection between shoe and insole should be checked for its electrical properties.

Additional information about insoles

The shoes are shipped with a removable insole. This means that the tests were performed with the insole inserted. **ATTENTION:** The shoes must only be used with the insole in place, and the insole supplied may only be replaced by a comparable insole from the original footwear manufacturer! Please contact our medimex customer service on telephone no. +49.6431.7302 300 if you need replacement insoles or if you have any other questions.

When an insole is inserted which was not approved by medimex, the shoe loses its safety certification!

We hope you enjoy the new, healthy wearing comfort of your mediPlugs⁺!

Explanation of the symbols you see on the shoes:

OB

Professional footwear – meets the basic requirements and has an additional protective function

A

Electrical contact resistance, antistatic footwear

E

Energy absorption in the heel area

SRB

Slip resistance on steel plate/glycerine

Additional slip resistance classes:

SRA

Slip resistance on ceramic tile floor/cleaning agent

SRC

Slip resistance on ceramic tile floor/cleaning agent and steel plate/glycerine

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mediPlogs+ is a registered trade mark.